



INSTRUCTIONS

AMERX[®] Wound Care Kit



HELIX3-CP[®] COLLAGEN POWDER

REMOVING THE DRESSING

Carefully remove the existing dressing/gauze that is covering the wound. To make the dressing easier to remove with little or no discomfort, you may dampen the dressing with AMERIGEL[®] Saline Wound Wash. Wait several minutes then remove carefully.

STEP 1: CLEANSING THE WOUND

Cleanse the wound with AMERIGEL Saline Wound Wash by spraying the affected area. The pressurized action of the Wound Wash requires only a small amount of spray to cleanse the wound. Gently blot the wound dry. It is not necessary to remove all collagen powder from the wound when cleansing.

STEP 2: APPLYING HELIX3-CP COLLAGEN POWDER

Following wound cleansing, lightly sprinkle enough HELIX3-CP Collagen Powder to cover the wound surface in a layer approximately 1/4 inch thick. **DO NOT PACK THE POWDER TIGHTLY WHEN APPLYING.** (The product will expand as it absorbs wound fluids.)



STEP 3: APPLY THE AMERX BORDERED GAUZE DRESSING

Remove the AMERX Bordered Gauze dressing from the package. Remove one side of the paper liner. Place dressing over the wound and surrounding skin. Remove the remaining liner. Carefully smooth the edge of the dressing to ensure proper contact with surrounding skin.

CHANGE THE DRESSING DAILY OR AS INDICATED BY YOUR HEALTH CARE PROVIDER.

WARNINGS:

- Due to risk of infection from loss of sterility, discard any remaining product following initial application.
- **DO NOT** use HELIX3-CP if you are allergic or sensitive to materials of bovine origin.
- Discontinue the use of HELIX3-CP and notify your doctor if excessive redness, pain, swelling or blistering occurs.

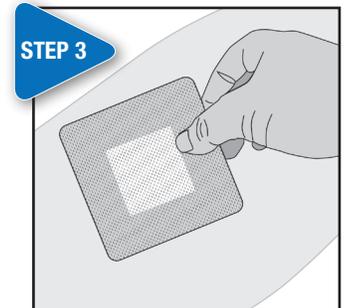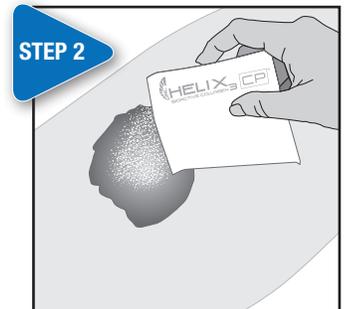

**FOR ANY QUESTIONS OR CONCERNS,
PLEASE CONTACT YOUR PHYSICIAN**

*Attention:
Please place business card
here before making copies.*

Your next scheduled visit:

Date: _____

Time: _____ **AM / PM**